



120.00 mm

Adjustable Lancing Device with Ejector

Back View

Front View

NOT FOR HUMAN USE.
For use on cats and dogs only.
Lancing device is used with disposable blood lancet to collect blood from pinna or paw for blood tests.

CAUTION:
Always use a new, sterile lancet. Lancets are for single use only. Reused lancets can dull or bend the tip of the lancet, causing damage to the skin, scarring and greater pain.
For single pet use, do not share with another animal.
DO NOT store lancet in lancing device.
DO NOT use lancets beyond the use-by date.
PETRACKR® is not a toy. Keep the PETRACKR® Lancets and Lancing Device out of reach of children and pets.

Please carefully read all instructions before testing
PETRACKR® lancing device has 5 penetration depths.
Turn Depth Dial to desired lancing depth.
Lancing Depth Guide: number 1 for soft / thin skin, number 3 for average skin, number 5 for thick / calloused skin, and finer adjustments can be made by setting arrow between numbers.

Cleaning and disinfecting

Clean the surface of the lancing device with a soft cloth dipped with clean water. Please don't immerse the body of lancing device into any liquid to avoid damage. The surface of the lancing device can be wiped and disinfected with 75% ethanol. If there is blood stain inside the cap of lancing device, turn the Depth Dial to arrow "→" position and pull out the Depth Dial to wipe and clean the blood stain with 75% ethanol (as shown in the figure). After disinfection, align the arrow mark on the Depth Dial with the depth indicator arrow and install the Depth Dial back to the End Cap.

UNIVERSAL BIOSENSORS PTY LTD,
1 Corporate Avenue,
Rowville, Victoria, 3178 Australia
www.universalbiosensors.com
Petrackr@universalbiosensors.com
© 2022 Universal Biosensors Pty Ltd
PETRACKR® and the PETRACKR logo are
registered trademarks of Universal
Biosensors. All rights reserved.

79210v1.3.1
Revision date: 04/2025

80.00 mm

120.00 mm

Instructions for Use

1. Wash hands with soap and clean water, rinse and dry thoroughly. Twist off the End Cap of the Lancing Device.

2. Push Lancet firmly into Holder.

3. Remove Lancet Protective Cover.
Note: Save the Protective Cover for safe disposal of used Lancet.

4. Adjust to the suitable depth.

5. Gently pull Priming Barrel until a click is heard.

6. Place the End Cap firmly against pinna or paw pad. Press Trigger Button.

7. Remove End Cap and use the saved Lancet Protective Cover to recap Lancet.

8. Push forward the Ejector to dispose of the used Lancet in appropriate container.

Batch code

Manufacturer

Caution

Consult instructions for use

Expiry Date

www.universalbiosensors.com

80.00 mm

<div></div> <div>Universal Biosensors Pty Ltd 1 CORPORATE AVE, ROWVILLE, VIC 3178, AUSTRALIA TEL: +61 3 9213 9000 FAX: +61 3 9213 9099</div>		
TITLE: PETRACKR Lancing Device IFU		
DRAWING NO.: 79210	VERS. 1.3.1	
DRAWN DATE: 17-April-25		
SCALE: 1:1	SHEET 1 OF 1	SHEET SIZE A5
Commercial in Confidence © Copyright 2022 Property of Universal Biosensors Pty Ltd		

Scale:1:1	Colour: <div><div></div>Die<div></div>CMYK: 0/0/0/100</div>	NOTES:
-----------	---	--------